


EXISTING CEILING HEIGHT IS 9'-7 1/2"

NOTE: THE UPS IS SUPPLIED AND DELIVERED TO CUSTOMER'S LOADING DOCK BY SIEMENS. CUSTOMER'S ELECTRICIAN IS RESPONSIBLE FOR MOVING FROM LOADING DOCK TO FINAL LOCATION AND COMPLETING ALL CONNECTIONS. THE UPS MUST NOT BE LOCATED IN A PATIENT ENVIRONMENT. SIEMENS PROJECT MANAGER WILL SCHEDULE UPS STARTUP PRIOR TO DELIVERY OF SIEMENS IMAGING EQUIPMENT.

THIS LAYOUT IS REFLECTIVE OF THE LATEST SALES CONFIGURATION. ANY CHANGES TO THIS SALES CONFIGURATION MAY REQUIRE A REVISION TO THIS PROJECT PLAN.

SINKS, COUNTERTOPS AND ALL CASEWORK SHOWN IS SUGGESTED AND MUST BE DESIGNED SUPPLIED AND INSTALLED BY CUSTOMER/CONTRACTOR.

IT IS THE RESPONSIBILITY OF THE CUSTOMER/CONTRACTOR TO PROVIDE A MEANS OF MOUNTING THE SYNGO X PC TOWER OFF OF FINISHED FLOOR FOR DAMAGE PROTECTION AGAINST TIP-OVER, FLUIDS, IMPACT, ETC.

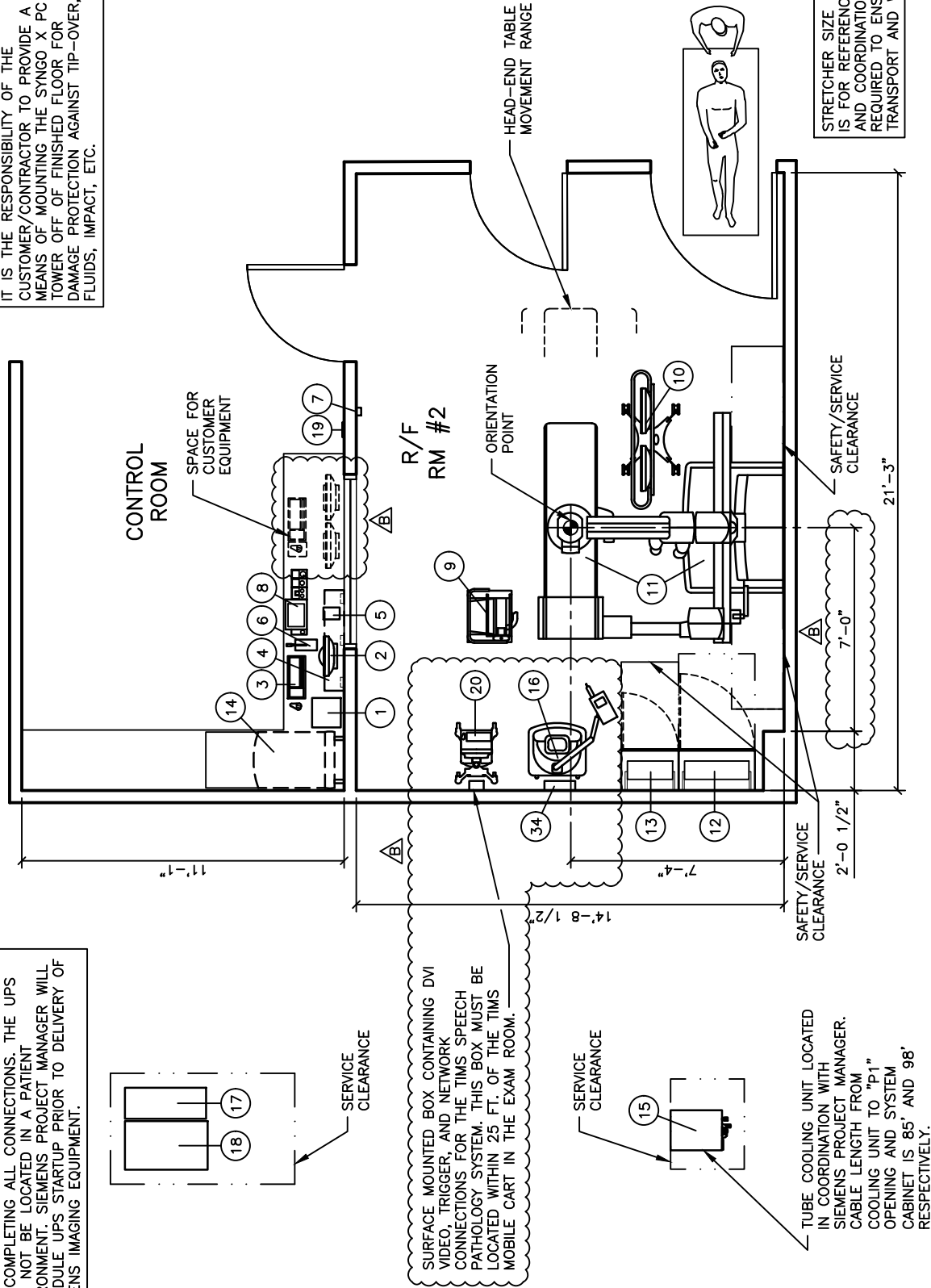

Click to acquire Typical PDF technical data

EQUIPMENT LEGEND

NO	DESCRIPTION	SMS SYM	WEIGHT (LBS)	BTU/HR TO AIR	DIMENSIONS (INCHES)			REMARKS
					W	D	H	
1	ACE (ARCHIVE CONTROL EXTENSION)	⊖	13	N/A	12 1/4	11 3/4	4	MTD. ON CONTROL COUNTER
2	CONTROL ROOM DISTRIBUTOR	⊖	64	342	41 1/2	8 1/4	16 1/8	MTD. ON WALL
3	KEYBOARD	⊖	2.2	342	17 1/2	6 1/8	2 1/8	MTD. UNDER COUNTER OR ON CONSOLE
4	19" LIVE DISPLAY	⊖	15	256	16 1/2	8 1/4	13 1/2	ON COUNTER OR CONSOLE
5	INTERCOM POWER UNIT	⊖	---	---	6 3/4	5	1 3/8	ON COUNTER
6	INTERCOM MICROPHONE/LOUDSPEAKER (CONTROL ROOM)	⊖	---	---	4 1/2	9	2	ON COUNTER
7	INTERCOM LOUDSPEAKER (PROCEDURE ROOM)	⊖	---	---	3 1/4	2	6	WALL MOUNTED
8	2ND. TABLE CONTROL MODULES	⊖	16	---	20	8 3/4	3 1/2	ON COUNTER
9	TROLLEY FOR CONTROL MODULES	⊖	59	---	23	21	40	MOUNTED ON CASTERS
10	MONITOR CART WITH (2) 19" FLAT SCREEN DISPLAY	⊖	---	---	---	---	---	MOUNTED ON CASTERS
11	ARTIS ZEE MULTI-PURPOSE UNIT	⊖	4,469	---	---	---	---	FLOOR MOUNTED
12	POLYDOROS A100 (POWER UNIT 1)	⊖	662	3,413	31 1/2	17 1/8	87	FLOOR MOUNTED
13	SYSTEM CONTROL CABINET	⊖	594	5,460	23 1/2	17 1/8	87	FLOOR MOUNTED
14	AXIS IMAGE SYSTEM	⊖	331	4,347	23 3/4	37 1/4	28	MTD. ON CASTERS
15	TUBE COOLING UNIT	⊖	80	15,355	16 1/2	28 1/4	19 1/4	FLOOR OR SHELF MOUNTED
16	ANGIOMAT ILLUMENA INJECTOR INTEGRATED PEDESTAL MOUNTED	⊖	90.5	---	---	---	---	SEE MFG REQUIREMENTS
17	EATON 9355 15KVA UPS AND BATTERY	⊖	755	8,134	12 3/4	33 1/2	47 3/4	SEE MFG REQUIREMENTS
18	EATON 9355 OUTPUT TRANSFORMER CABINET	⊖	490	---	20	34 1/8	66	SEE MFG REQUIREMENTS
19	EATON 9355 REMOTE MONITORING DEVICE	⊖	0.5	---	6	1	3	SEE MFG REQUIREMENTS
20	TIMS MOBILE CART W/ MONITOR, MIC, KEYBOARD & MOUSE	⊖	220	512	24	25	64 - 72	
21	ITEM REMOVED							
22	ITEM REMOVED							
23	ITEM REMOVED							
24	ITEM REMOVED							
25	ITEM REMOVED							
26	ITEM REMOVED							
27	ITEM REMOVED							
28	ITEM REMOVED							
29	ITEM REMOVED							
30	ITEM REMOVED							
31	ITEM REMOVED							
32	ITEM REMOVED							
33	ITEM REMOVED							
34	INJECTOR WALL CONNECTION BOX	⊖	11	---	12 3/4	4	10 1/2	WALL MOUNTED



ARCHITECTURAL NOTES

- 1) ALL PRELIMINARY EQUIPMENT LAYOUTS SUBMITTED BY SIEMENS HEALTHCARE ARE BASED ON THE RECOMMENDED SPACE NECESSARY FOR THE OPERATION AND SERVICEABILITY OF THE EQUIPMENT BEING PROPOSED. SIEMENS WILL NOT SUBMIT AN EQUIPMENT LAYOUT THAT IS NOT IN THE BEST INTEREST OF BOTH THE CUSTOMER AND SIEMENS. ALL EQUIPMENT LAYOUTS ARE BASED EITHER ON AN ACTUAL SITE SURVEY OR ARCHITECTURAL DRAWINGS SUPPLIED TO SIEMENS. SIEMENS WILL NOT BE RESPONSIBLE FOR ANY ALTERATIONS THAT ENCROACH WITHIN DESIGNATED SAFETY AND SERVICE CLEARANCE ZONES AS INDICATED ON DRAWINGS (I.E., PIPE CHASES, VENTILATION DUCTS, CASEWORK, AND SOFFITS, ETC.) MADE BY THE CUSTOMER OR REQUIRED BY A CUSTOMER'S ARCHITECTURAL FIRM ONCE PRELIMINARY DRAWINGS HAVE BEEN SUBMITTED AND APPROVED. DO NOT ALTER ANY SPECIFICATIONS AND/OR DIMENSIONS WITHOUT CONTACTING AND RECEIVING WRITTEN CONFIRMATION FROM SIEMENS PROJECT MANAGER.
- 2) SIEMENS HEALTHCARE IS NOT AN ARCHITECTURAL OR ENGINEERING FIRM. DRAWINGS SUPPLIED BY SIEMENS ARE NOT CONSTRUCTION DRAWINGS. THEREFORE, THESE DRAWINGS ARE TO BE USED ONLY FOR INFORMATION TO COMPLEMENT ACTUAL CONSTRUCTION DRAWINGS AVAILABLE FROM A CUSTOMER APPOINTED ARCHITECTURAL REPRESENTATIVE OR A CUSTOMER'S ENGINEERING DESIGN GROUP. THE CUSTOMER'S ARCHITECT AND GENERAL CONTRACTOR SHALL BE ULTIMATELY RESPONSIBLE FOR COMPLIANCE WITH ALL APPLICABLE CODES AND PROFESSIONAL DESIGN REQUIREMENTS INCLUDING OSHA/NEC SAFETY CLEARANCE REQUIREMENTS IN ADDITION TO SIEMENS-REQUIRED SAFETY/SERVICE CLEARANCES SHOWN.
- 3) THE CUSTOMER IS RESPONSIBLE FOR ALL ROOM AND AREA PREPARATION COSTS, PROFESSIONAL FEES, PERMITS, REPORTS, AND INSPECTION FEES.
- 4) EQUIPMENT WARRANTIES, EXPRESSED OR IMPLIED ON THE PART OF SIEMENS SHALL BE CONTINGENT UPON STRICT COMPLIANCE WITH THE ARCHITECTURAL, STRUCTURAL, ELECTRICAL, MECHANICAL AND RECOMMENDATIONS AND REQUIREMENTS CONTAINED IN THESE DRAWINGS, UNLESS SPECIFIED OTHERWISE.
- 5) ALL DIMENSIONS SHOWN ARE FROM FINISHED SURFACES UNLESS SPECIFIED OTHERWISE.
- 6) THIS DRAWING DOES NOT PROVIDE RADIATION SHIELDING REQUIREMENTS FOR X-RAY AND ASSOCIATED EQUIPMENT. THE CUSTOMER IS RESPONSIBLE FOR CONSULTING WITH A REGISTERED RADIATION PHYSICIST. ACTUAL PROTECTION REQUIREMENTS SHALL BE SPECIFIED BY A REGISTERED RADIATION PHYSICIST AT CUSTOMER'S ENGAGEMENT AND EXPENSE. RESPONSIBILITY FOR ALL INFORMATION AS TO THE ROOM LOCATION, USE AND NUMBER OF ANTICIPATED EXAMINATIONS TO BE PERFORMED PER TIME PERIOD SHALL BE PROVIDED TO THE PHYSICIST BY THE CUSTOMER. THE CUSTOMER SHALL FURTHER TAKE ALL RESPONSIBILITY IN THE COMMUNICATION AND COORDINATION OF ACTIVITIES OF THE RADIATION PHYSICIST AND THE ARCHITECTURAL REPRESENTATIVE.
- 7) SIEMENS HEALTHCARE SHALL BE RESPONSIBLE FOR SIEMENS EQUIPMENT INSTALLATION, CALIBRATION, CONNECTION AND INSTALLATION OF SIEMENS PROVIDED CABLES. THE CUSTOMER/ELECTRICAL CONTRACTOR IS RESPONSIBLE FOR TERMINATIONS OF CUSTOMER/ELECTRICAL CONTRACTOR-SUPPLIED CABLES TO SIEMENS EQUIPMENT. IN THE EVENT THAT SPECIFIC TRADE RULES OR LICENSE REQUIREMENTS PROHIBIT THIS, THE CUSTOMER SHALL INITIATE THE SERVICES OF APPROVED OTHER CONTRACTORS AND PAY FOR SELECTED, APPROVED PARTIES TO PERFORM THIS WORK WITH SUPERVISION PROVIDED BY SIEMENS. CALIBRATION WHEN ACCOMPLISHED OUTSIDE OF NORMAL INSTALLATION SEQUENCES DUE TO CONTRACTOR OR TRADE RULE ACTIONS OR REQUIREMENTS SHALL BE SUPPORTED BY, CHARGED TO, AND ACCEPTED BY THE CUSTOMER AS AN ADDITIONAL INSTALLATION EXPENSE.
- 8) THE CUSTOMER SHALL COORDINATE WITH SIEMENS PROJECT MANAGER THE LOCATIONS AND TRAVEL OF ALL ANCILLARY EQUIPMENT TO BE CEILING OR WALL MOUNTED (I.E.: O.R. LIGHTS, MEDICAL GAS COLUMNS, PHYSIOLOGICAL MONITORING INJECTORS, CRT PLATFORMS, SPRINKLER HEADS, SMOKE DETECTORS, ELECTRICAL OUTLETS, HVAC GRILLES, SPEAKERS, AND GENERAL ROOM LIGHTING, ETC.).
- 9) THE GENERAL CONTRACTOR/CUSTOMER SHALL BE RESPONSIBLE FOR ALL FINAL PAINT, TOUCH-UP AND ANY COSMETIC OR TRIM WORK WHICH NEEDS TO BE OR IS REQUIRED TO BE COMPLETED AFTER THE INSTALLATION OF THE SIEMENS EQUIPMENT AND ANY ASSOCIATED SUPPORT APPARATUS.
- 10) CUSTOMER/CONTRACTOR MUST ASSIST SIEMENS INSTALLERS WITH INSTALLATION OF EQUIPMENT ABOVE 14'-0". REFER TO THE ELECTRICAL NOTES ON SIEMENS SHEET E-101 FOR MORE DETAILS.

FOR ADDITIONAL PLANNING RELATED QUESTIONS, PLEASE REFER TO TYPICAL DRAWING SET OR SIEMENS PROJECT MANAGEMENT.

EXAM RESTRICTIONS:
NOT APPLICABLE

WARNING NOTES:
NOT APPLICABLE

MISCELLANEOUS NOTES:
NOT APPLICABLE



Authorization:

I have reviewed the below referenced plan(s) and understand and agree with the proposed room configuration and equipment arrangement. By signing below, I agree that I have the authority to sign on behalf of the customer. I have been fully informed by Siemens about the limitations (if applicable) that exist regarding the proposed equipment installation. Siemens has explained that the listed limitations may interfere with certain functions of the equipment. We agree to assume responsibility to inform employees, agents, contractors, users or any other persons handling this equipment of these limitations and will assure that the equipment is handled in compliance with these parameters.

Signature: _____ Date: _____

Name Printed: _____ Title: _____

PROJECT MANAGER: MATT WIED
PHONE: (214) 263-4318
EMAIL: MATTHEW.WIED@SIEMENS-HEALTHINEERS.COM

EL PASO COUNTY HOSPITAL DISTRICT

4815 ALAMEDA AVE, 9159, EL PASO, TX 79905

SIEMENS

B 08/22/23 REMOVED YSIO COMPONENTS

SHEET OF
2 2

DATE:
08/22/23

DRAWN BY:
B. CLEATON

REF:
CPQ-781107

R/F RM #2 - ARTIS ZEE MULTI-PURPOSE WITH TIMS
REFERENCE DOCUMENT - NOT FOR CONSTRUCTION

2310949